INgezim® PPA Compac 2.0

R.11.ASF.K3

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INgezim® PPA Compac 2.0

R.11.ASF.K3

INgezim® PPA Compac 2.0 is a blocking immunoenzymatic assay based on the use of a monoclonal antibody specific of VP72 protein of the African Swine Fever Virus (ASFV) and a recombinant protein VP72 as an antigen.


KIT FEATURES

APPLICATION

Detection of antibodies specific of VP72 protein in pig serum (domestic pig, wild boar, etc.).

TECHNICAL BASIS

- The plates are supplied coated with VP72 recombinant protein of ASFV. Samples are added to the wells and incubated.
- If the samples contain ASFV-specific antibodies, they will bind to the antigen.
- When the conjugate (VP72 protein-specific monoclonal antibody, peroxidase-labeled, mAb-PO) is added, it will bind to the protein only if there are no antibodies from the sample blocking the antigen (negative animals). In case there are antibodies blocking the antigen (infected or vaccinated animals), the conjugate will not be able to bind to it. This binding is revealed by colorimetric reaction after substrate addition.

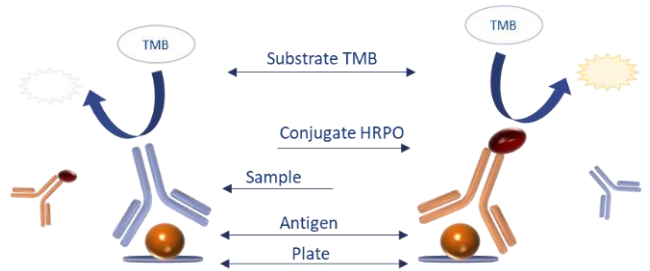INTERPRETATION OF THE RESULTS

The test establishes 2 cut-offs, which will classify the samples as **Positive** or **Negative**, depending on the value of the optical density of the sample in the test.

ASSAY VALIDATION

DIAGNOSTIC SENSITIVITY

To determine the diagnostic sensitivity, the following samples were analyzed, obtaining these results:

SAMPLES	RESULTS
70 sera from experimentally infected pigs	-80% days 10-13 p.i. -93.75% days 17-21 p.i. -100% from day 21 p.i.
587 field serums of pigs and wild boar (previously classified by IPT as positive)	-46% for IPT 1/1280 titers (samples corresponding to days 7-10 p.i.) -61% for IPT 1/2560 securities -83% for IPT 1/5120 titers (samples corresponding to days 12-14 p.i.) -96% for IPT 1/10240 titers (samples corresponding to 2nd week p.i.) -100% for IPT $\geq 1/40960$ / for IPT titres 1/40960

DIAGNOSTIC SPECIFICITY

To determine the diagnostic specificity, the following samples were analyzed, obtaining these results:

SAMPLES	RESULTS
85 samples of field serum from pigs and wild boar, negative IPTs	-96.5% correspondence with IPT
1030 samples from Spanish or Dutch farms +125 samples of wild boar in free areas of Spain.	>99% free zones
1240 (1030+125+85) total samples	Overall value of diagnostic specificity for all sera analysed (1240) 99.7%

ANALYTICAL SENSITIVITY

To determine the diagnostic sensitivity, the following samples were analyzed, obtaining these results:

SAMPLES AND OBJECTIVE	RESULTS
CISA-INIA Reference Serum (For detection limit)	Detects positive reference serum CISA-INIA-ASF-CP lot C+120 at dilution 1/8192
169 (118 IPT positive) sera from domestic pigs experimentally infected with genotypes I, II, X, and XXIII (For early detection)	94.9% with respect to IPT (False negatives corresponding to early stages, 7-21 d.p.i., with IPT $\leq 1/320$ titers)
237 sera from domestic pigs experimentally infected with genotypes I, II, X, and XXIII	For detection of antibodies during infection -68.75% at 10-15 p.i. days -80% to days 15-21 p.i. -99.39% days after 21 p.i. For correspondence with IPT titles -40% shares $\leq 1/1280$ -100% titles 1/2560 -94% titles 1/5120 -100% titles $\geq 1/10240$ Overall analytical sensitivity value for IPT titers $\geq 1/2560$, 99%
406 (169+237) sera infected with genotypes I, II, X, and XXIII (for inclusivity)	Capable of detecting at least antibodies against genotypes I, II, X, and XXIII

ANALYTICAL SPECIFICITY

SAMPLES	RESULTS
65 pig and wild boar sera positive for other agents	-No cross-reaction with other agents

COMPOSITION OF THE KIT

- 96-well microtiter plates
- Vials with Positive Control
- Vials with Negative Control
- Vials with Conjugate
- Bottle with Washing Solution
- Bottle with Diluent
- Bottle with Substrate (TMB)
- Bottle with Stop Solution

Registration nº
EXPIRATION: 24 MONTHS
Stored at 2°C-8°C

GOLD STANDARD DIAGNOSTICS MADRID, S.A.
C/Hermanos García Noblejas 39, 8º
28037 MADRID (SPAIN)
Tel: (+34)91 3680501
<http://www.goldstandarddiagnostics.com/>