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<http://www.goldstandarddiagnostics.com/>

INgezim® PRRS BLUE is based on an indirect ELISA technique, which uses an A/G protein conjugate with HRPO, and the recombinant N proteins of American and European strains of PRRS as antigen.

KIT FEATURES

APPLICATION


Detection of antibodies specific for both, American and European strains of PRRSV in porcine serum and plasma samples.

TECHNICAL BASE

1. Plates are supplied coated with PRRSV antigen1. On these wells, samples are added and incubated.
2. If the samples contain specific antibodies to PRRSV, they will bind the antigen.
3. When the conjugate (A/G protein-HRPO) is added, it will bind to the latter. This binding is detected through a colorimetric reaction after substrate addition.

RESULTS INTERPRETATION

The assay establishes one Cut Off value measured in **S/P** which will classify the samples as **Positive** or **Negative**, depending on the value of the optical density of the sample in the assay.
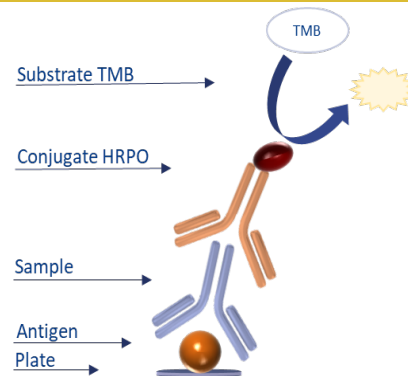


ASSAY VALIDATION


ANALYTICAL SENSITIVITY

Assay precocity at day 7 p.i.: 30 samples collected at day 7 p.i. experimentally infected with European strains were analyzed.

The assay was performed in comparison with the IDEXX PRRS X3 Ab commercial kit. The results obtained in this study indicated that the assay is able to detect antibodies specific to European strains of PRRSV at day 7 p.i. in 57% of the cases and is related to the responsiveness of each animal.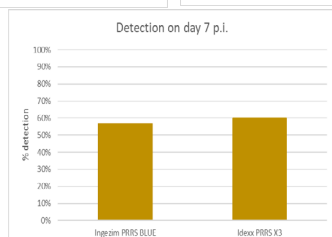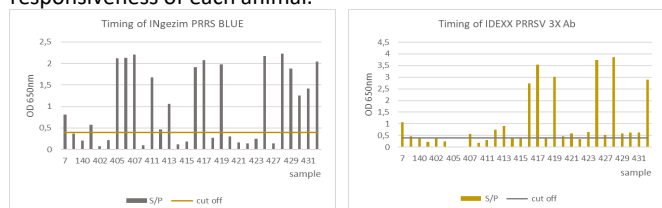

Assay precocity at different days p.i.: Samples from 6 pigs experimentally infected with the European strain were analysed and extractions were performed at days 0, 4, 14, 20 and 21 p.i. The assay is able to detect specific antibodies to European strains of PRRSV from day 14 p.i. onwards.

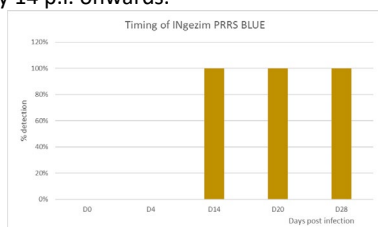Ability to detect specific antibodies of American and European strains:

17 samples from pigs infected with American strains and 9 piglets PCR positive to both strains were tested.

The obtained results indicated that the assay can detect antibodies specific to American strains of PRRSV.

ANALYTICAL SPECIFICITY

Sera from animals infected with other agents that affect swine (dysentery, brucella and tuberculosis) were analyzed. The results indicated that there is no cross reaction with antibodies for other diseases affecting pigs.

DIAGNOSTIC SENSITIVITY

643 sera from animals previously classified as positive by the IDEXX PRRS X3 Ab commercial assay were analyzed. INgezim® PRRS BLUE has a PPA or relative sensitivity with respect to IDEXX PRRS X3 Ab of 95.3%.

DIAGNOSTIC SPECIFICITY

137 samples were tested: 50 samples from young sows (gilts) from PRRSV-free Polish farms and 87 samples from unvaccinated and PCR-negative wild boar from Germany. The obtained results indicated a diagnostic specificity higher than 99%.

EXTERNAL STUDIES (end user)

Study 1: (Jedryczko VET Lab Group, Poland). Three groups of sera expected to be serologically positive were tested with INgezim® PRRS BLUE and qPCR and retested with IDEXX X3. The obtained results indicated a PPA of 100% and NPA of 99%.

Study 2: (Tiergesundheitsdienst Bayern e.V., Germany). 340 sera of swine and wild boar of different ages and vaccination status. All the samples were analyzed using INgezim® PRRS BLUE and IDEXX PRRS X3 Ab. The results obtained in this study indicated a PPA of 97% and a NPA of 94%.

Study 3: (China). 140 swine sera previously tested using IDEXX PRRS X2 Ab and 54 swine sera previously tested using IDEXX PRRS X3 Ab, were analyzed. The obtained results indicated a PPA of 100% and NPA of 98%.

KIT COMPOSITION

- 96 well microtitration plates
- Vials with Positive Control
- Vials with Negative Control
- Vials with Conjugate
- Bottles with Washing Solution
- Bottles with Substrate (TMB)
- Bottles with Stop Solution
- Bottles with Diluent

Spanish registration nº12003-RD
EXPIRATION: 24 MONTHS
Stored at 2°C-8°C

GOLD STANDARD DIAGNOSTICS MADRID, S.A.
C/Hermanos García Noblejas 39, 8º
28037 MADRID (SPAIN)
Tel: (+34)91 3680501

<http://www.goldstandarddiagnostics.com/>