INgezim® ASFV CROM Ab 2.0

Prod Ref: 11.ASF.K41

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TECHNICAL BASIS

INgezim® ASFV CROM Ab 2.0 is a qualitative lateral flow or immunochromatographic assay based on the use of colloidal gold nanoparticles bound to a recombinant form of the viral protein VP72 of African Swine Fever Virus (ASFV) or to a control protein. The membrane contains a test line region (T) in which that same form of the VP72 is immobilised and a control line region (C) in which an antibody to the control protein is immobilised. If the sample contains antibodies to ASFV, they will interact with the protein bound to the gold nanoparticles. After adding the running buffer, these gold-protein-antibody complexes will migrate through the membrane and they will bind to the VP72 protein immobilised in the T region, resulting in the appearance of a reddish coloured line. In case no antibodies are present in the sample, no colour will appear in that region. In both cases, colloidal gold nanoparticles covered with the control protein will continue to migrate through the membrane and they will be recognised by the monoclonal antibody immobilised in the control line region. For this reason, the appearance of a red line in the control line region (C) indicates that the assay developed correctly.

PRECAUTIONS AND WARNINGS FOR USERS

- Carefully read the instructions for use prior to analyse any sample.
- Let the reagents reach room temperature before their use.
- Do not mix reagents or instructions from different kits or lots.
- Avoid any potential contamination of the reagents.
- Do not remove the lateral flow device from its original packaging until the moment just prior to analysis.
- Do not use the kits after the expiry date.
- Do not eat, drink, or smoke while samples or kit reagents are being handled.
- Do not pipette by mouth.

STORAGE OF COMPONENTS

All reagents must be kept between 4 and 25 °C in their original packaging until use. **WARNING! DO NOT FREEZE ANY KIT COMPONENT.**

PREPARATION OF SAMPLES

The test can be performed using the following samples collected from swine (pigs or wild boar):

- Fresh blood samples. Analysis must be performed right after collection of the sample.
- Blood samples collected by venous puncture. In these cases, the use of anticoagulant compounds (EDTA, heparin...) in the collection is recommended to avoid the formation of clots that may interfere with the assay. These samples can be stored at 4 °C until use for a maximum period of 48 hours.

Before doing the analysis, let the refrigerated blood samples reach the room temperature for an appropriate result.

NOTE: *The blood matrix may have a significantly lower sensitivity than the serum matrix.*

TEST PROCEDURE

- **Sample addition:**

Blood samples

Using the plastic micropipettes provided in the kit, take 20 µL of blood by filling the conduct up to the first teat (see picture 1.1.) and add them into the device's quadrangle window marked as "S". For blood samples, it is particularly important to wait 1 min to the complete absorption of the sample.

Serum samples

Take serum sample using the disposable 10 µL pipettes provided in the kit. Volume will be adjusted itself by capillarity. Add the serum sample into the device's quadrangle window marked as "S". Wait 1 min to the complete absorption of the sample. Running buffer addition: Add into the oval window marked as "B", one by one, 5-6 drops of the running buffer contained in the dropper (see picture 2).

READING AND RESULT INTERPRETATION

Read the results of the test after 15 minutes from adding the running buffer.

NEGATIVE RESULT. Only one red line appears in the region marked as "C" (Control). **POSITIVE RESULT.** Two lines appear, one on the C region (control) and another one on the T region (test). Line intensities can vary from faint to highly intense; all should be considered positive.

INVALID RESULT. No line appears in the control region, no matter if a line appears at the T region or not. Assay must be considered invalid, and analysis should be repeated with a new device.

Results read before the 15 min incubation time or after 30 min should not be considered valid.

WARNING: *The blood matrix may have a significantly lower sensitivity than the serum matrix.*

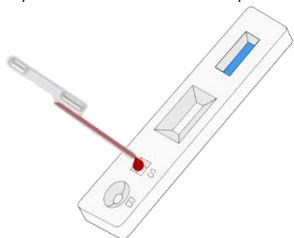
PROCEDURE SCHEME

1.1.- Sample addition

Fill the micropipette up to the first teat.


Add the 20 µL of blood into the device's Sample-window (S).

1.2. Sample addition

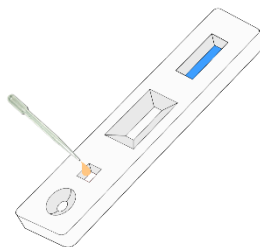
Take 10 µL of serum with the fixed volume pipette and add them into the device's Sample-window (S).

Wait at least 1 min until the complete absorption of the sample.


2.- Running buffer addition

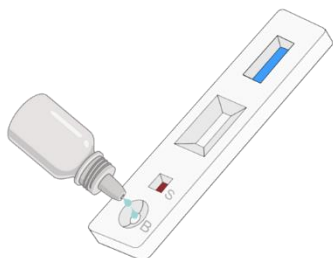Add 5-6 drops one by one, in the device's Buffer-window (B) using the dropper.

3.- Results reading

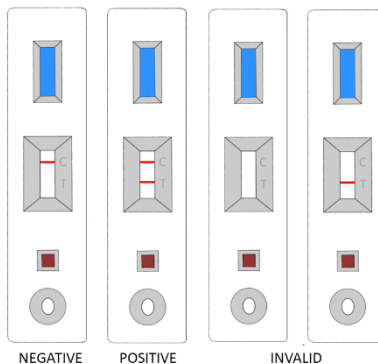Wait 15 min and read the assays' results.

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