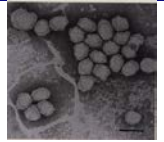
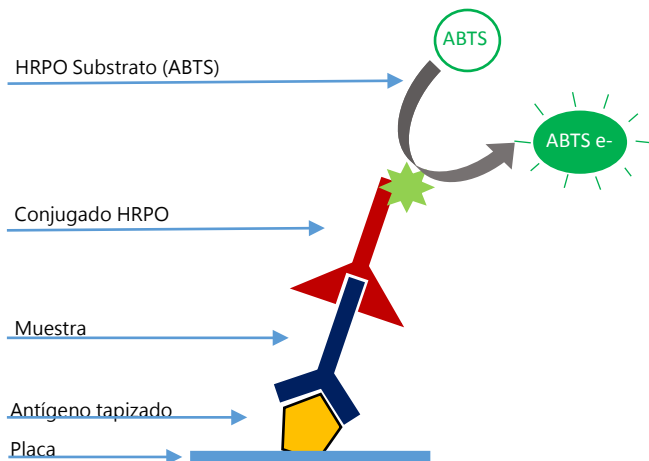
INgezim PRRS UNIVERSAL

R.11.PRU.K1

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
INgezim PRRS UNIVERSAL

R.11.PRU.K1

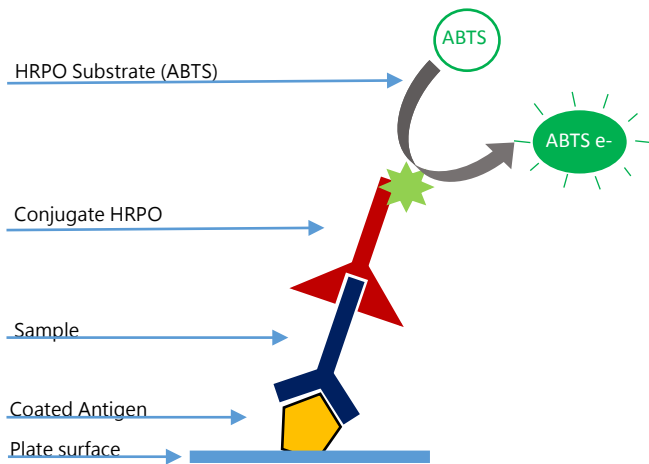INgezim PRRS UNIVERSAL is based on an indirect ELISA technique, which uses a monoclonal antibody (MAb) specific for porcine immunoglobulines (IgG), and a recombinant antigen (N protein of American and European strains of PRRSV).

TECHNICAL BASIS OF THE KIT

1. Plates are coated with PRRSV antigen (recombinant Vp7 protein from European and American strains). Sera samples are added and incubated.
2. If the samples contain specific antibodies to PRRSV, they will bind to the antigen.
3. When a MAb-PO specific of porcine immunoglobulines is added, it will bind to the IgG previously bound to the antigen. The binding is detected by the development of a colorimetric reaction after the addition of the substrate.

APPLICATION

Detection and/or titration of specific antibodies to PRRSV in porcine sera samples. The assay has been designed for the detection of antibodies specific for both American and European strains of PRRSV

INTERPRETATION OF THE RESULTS

There is one Cut Off value for the results interpretation. Samples with an OD higher than the Cut Off Value are positive and samples with an OD lower than the Cut Off value are negative.

VALIDATION

SENSITIVITY

1. Using experimentally vaccinated animals

A set of sera from different extractions taken from 4 experimentally vaccinated animals were analysed. Additionally, in the same study non-vaccinated animals were challenged and samples were taken. The results obtained show that INgezim PRRS Universal is able to detect specific antibodies to PRRSV at days around 8 and 12 post antigen's contact.

SPECIFICITY

A study using 250 European sera and 216 American sera was made. All these sera were catalogued as negative by other indirect assays available in the market. The results obtained indicated **97.2%** specificity analysing European samples and **93.2%** analysing American samples

USING FIELD SERA

A panel of 535 American sera was analysed and the results were compared with the Reference Technique IFA (Immunofluorescence Assay). The correspondence was 88%.

In the same assay, 830 European sera were compared with the reference technique in Europe IPMA (Immunoperoxidase Monolayer Assay). The correspondence was 100%

ONE WELL TITRATION

It is possible to obtain an approximated titre of the sera based on its OD value, by using a formula which corresponds with an exponential curve.

COMPOSITION OF THE KIT

- Microtitration plates of 96 wells
- Vials with Positive Control
- Vials with Negative Control
- Vials with Peroxidase Conjugate
- Bottle with Washing Solution
- Bottle with Diluent for sera
- Bottle with Diluent for conjugate
- Bottle with stop solution
- Bottle with substrate (ABTS).
- Bottle with Substrate Buffer

REGISTRATION NUMBER 1053 RD
PRODUCT MANUFACTURED BY INGENASA

SHELF LIFE: **18 months**
Stored at 2°C-8°C

Ed.020217