
INGENASA

INgezim BTV DR

R.12.BTV.K0

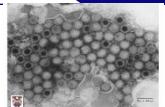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

INgezim BTV DR

R.12.BTV.KO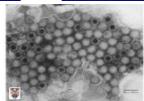

INgezim BTV DR is an immunoenzymatic assay which uses BTV's VP7 recombinant protein, both, to coat the plate and as conjugate.

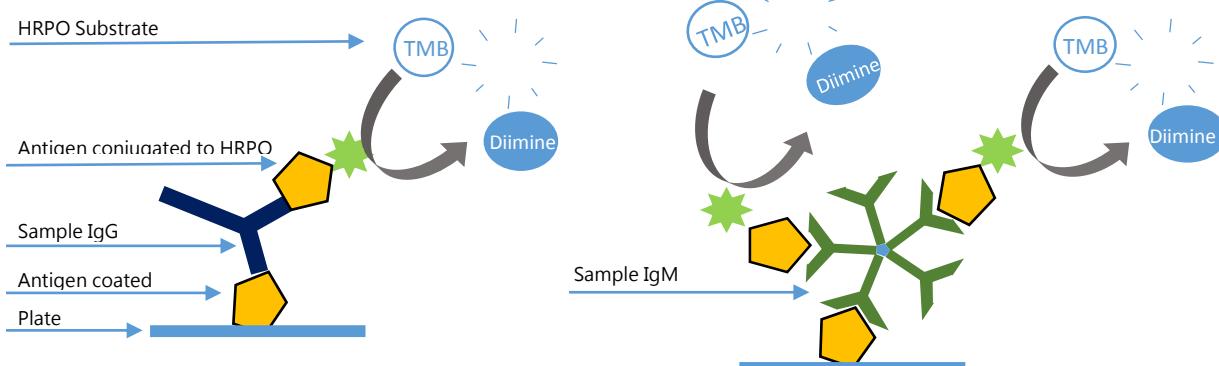TECHNICAL BASIS OF THE KIT

1. Plates are coated with BTV's VP7 recombinant protein. Sera samples are added to the wells and incubated.
2. If the sample contains specific antibodies to BTV, they will bind to the antigen.
3. When a HRPO conjugated recombinant VP7 protein is added, it will bind to the antibodies present in the sample if the serum sample is positive. This binding is detected by adding a specific substrate.

APPLICATION

Detection of specific antibodies to BTV's VP7 protein. The detection of antibodies is independent of the specie, being therefore able to use it in bovine, ovine or goat sera samples.

INTERPRETATION OF THE RESULTS

The assay establishes one cut off. Samples will be considered **Positive**, if its OD value is higher than the cut off; **Negative**, if its OD value is lower than the cut off.

VALIDATION

SENSITIVITY

1. Using OIE Reference Sera

The assay detects antibodies specific of the 24 known serotypes (Evaluated by the OIE Reference Laboratory, Pirbright).

2. Sensitivity in field trials.

A set of 172 sera previously classified as positive, were analyzed. The results obtained indicated that **sensitivity was 100%**.

3. Detection of late humoral response

Several studies using different vaccines from Syva and CZ (serotypes 1, 4 & 8) have been made. The results obtained indicate that the assay can detect specific antibodies to BTV at day 270 post vaccination.

SPECIFICITY

In order to determine the specificity of the assay, a set of 704 sera from herds catalogued as free of BTV were analyzed. **The results obtained indicated that the specificity was 99.8%.**

References: Lorca-Oró et al. PloS ONE May 2012; Vol 7 Issue 5 e36380; Lorca-Oró et al. Vet. Microbiol. (2012) 154 240-246; Lorca-Oró et al. Eur. J. Wildl. Res. DOI 10.1007/s10344-013-0783-y; Oura et al. Vaccine (2009) Dec 9;27(52):7326-7330.

COMPOSITION OF THE KIT

- Microtitration plates of 96 well
- Vials with Positive Control
- Vials with Negative Control
- Vials with Peroxidase Conjugate
- Bottle with Washing Solution
- Bottle with diluent
- Bottle with stop solution
- Bottle with substrate (TMB) ready to use

REGISTRATION NUMBER 1212 RD
PRODUCT MANUFACTURED BY INGENASA

SELF LIFE: 18 months
Stored 2°C-8°C

Ed.250118