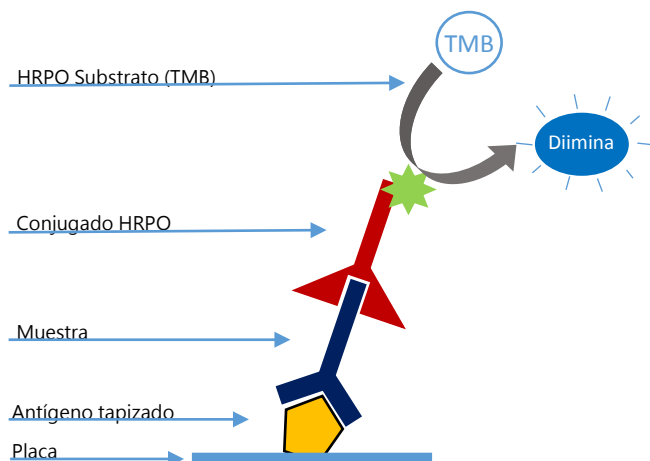
INgezim LEISHMANIA

R.15.LSH.K1

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REFERENCIAS:

Performance of commercially available serological diagnostic test to detect Leishmania infantum infection on experimentally infected dogs. Rodríguez-Cortés et al. Veterinary Parasitology 191:363-366 (2013)

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

INgezim LEISHMANIA

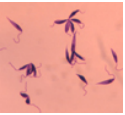R.15.LSH.K1

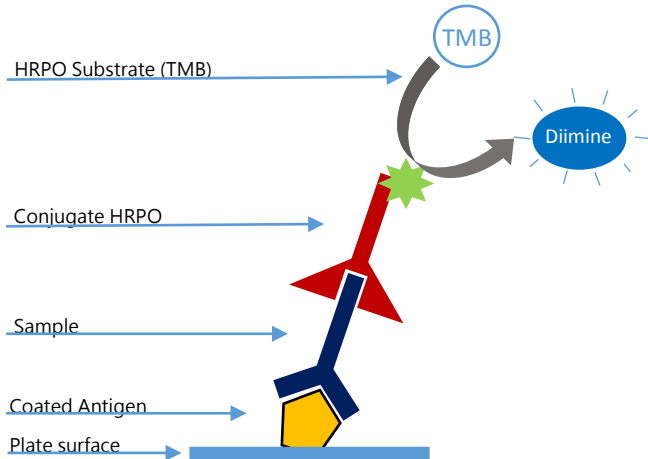Ingezim LEISHMANIA is based on an indirect enzymatic immunoassay (indirect ELISA), which uses a monoclonal antibody (MAb) specific to canine immunoglobulins (Igs) and an extract of *L. infantum* promastigote as antigen.

TECHNICAL BASIS OF THE KIT

1. Plates are coated with antigen of *L. infantum*. Serum samples are added and incubated.
2. If the samples contain specific antibodies to *L. infantum*, they will bind to the antigen.
3. When a MAb-PO, specific of canine immunoglobulins is added, it will bind to the Igs previously bound to the antigen. The binding is detected by the development of a colorimetric reaction after the addition of the substrate.

APPLICATION

Detection and/or titration of specific antibodies to *L. infantum* in canine sera samples.

INTERPRETATION OF THE RESULTS

Two cut offs are used for the results interpretation:

Screening: OD values higher than the positive cut off are considered **Positive**; OD values lower than the negative cut off are considered **Negative**. Samples with OD values in the range of both cut offs will be considered **Doubtful**.

Titration: The titre of each sample will be the last dilution at which the OD value is higher than the cut off.

SENSITIVITY AND SPECIFICITY

1. Study 1: The ELISA was initially developed by using a panel of 41 serum samples previously tested by IFA. Considering IFA results higher than 1/100 as positive and lower than 1/50 as negative, and not considering doubtful results, the ELISA yielded 95% correlation with IFA.

2. Study 2: A total of 795 canine sera previously classified by IFA were analyzed. The correlation between both assays was 80%.

REFERENCES

Performance of commercially available serological diagnostic test to detect *Leishmania infantum* infection on experimentally infected dogs. Rodríguez-Cortés et al. *Veterinary Parasitology* 191:363-366 (2013)

COMPOSITION OF THE KIT

- 96 wells microtitration plates
- Vials with Positive Control
- Vials with Negative Control
- Vials with conjugate
- Bottle with washing solution
- Bottle with diluent
- Bottle with substrate
- Bottle with stop solution

PRODUCT MANUFACTURED BY INGENASA

Shelf life: **18 months**
Store at: 2°C-8°C

Ed. 020217