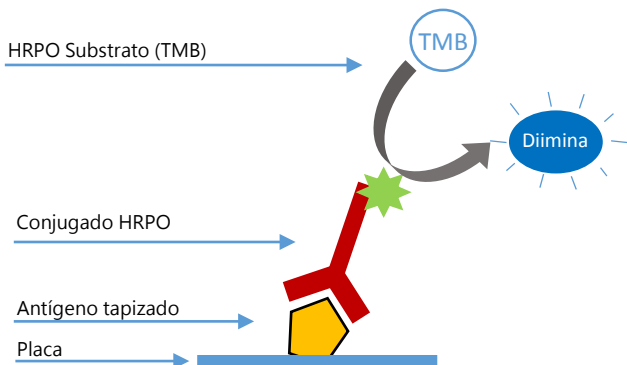
## INgezim PPR COMPAC

R.13.PPR.K3

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

## INgezim PPR COMPAC

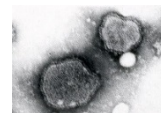R.13.PPR.K3

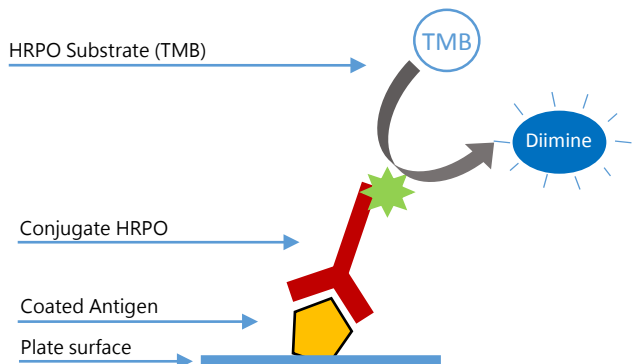**INgezim PPR COMPAC** is a blocking immunoenzymatic assay that uses the recombinant N protein of the Pest des Petites Ruminant Virus (PPRV) to coat the plate and a Monoclonal Antibody (MAb) specific to PPRV N protein as conjugate.

### TECHNICAL BASIS OF THE KIT

1. Plates are coated with the recombinant N protein of PPRV. Serum samples are added to the wells and incubated.
2. If the sample contains specific antibodies to PPRV, they will bind to the antigen.
3. Once a HRPO conjugated MAb is added, it will bind to the antigen only if there are no specific antibodies in the sample blocking the antigen coating the plate (negative). This binding is detected by adding a specific substrate.

### APPLICATION

Detection of specific antibodies to the N protein of PPRV. The detection of antibodies does not depend on the animal species, and can therefore be used with serum samples of all species affected by PPRV.

### INTERPRETATION OF THE RESULTS

Samples will be considered **Positive** (i.e. contains specific antibodies to PPRV), if their Blocking Percentage (BP) value is higher than or equal to 70%. Samples will be considered **Negative** (i.e. do not contain specific antibodies to PPRV), if their BP is lower than 70%.

## VALIDATION

### DIAGNOSTIC SPECIFICITY

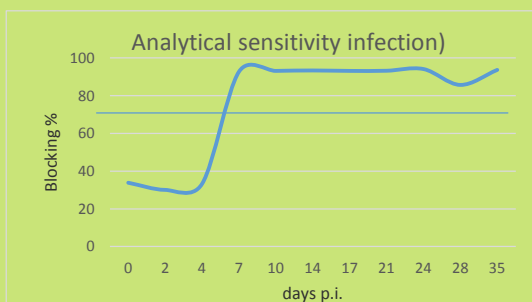
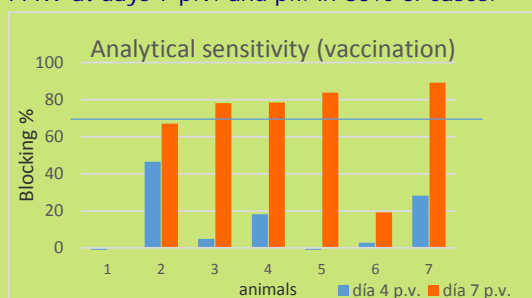
1066 animals (sheep & goats) from different PPRV free areas were analyzed. **The results obtained indicated specificity values higher than 99%.**

### ANALYTICAL SENSITIVITY

Sera of experimentally vaccinated and infected animals were analyzed:

- 7 one year old sheep, subcutaneously vaccinated with attenuated vaccine Nigeria75/1 (linage II). Extractions were made at days 4 and 7 p.v.
- 1 sheep one year old, intravenously inoculated with the field virulent strain Morocco 2008 (linage IV). Extractions at days 0, 2, 4, 7, 10, 14, 17, 21, 24, 28 & 35 p.i. were made.

The samples were analysed by INgezim PPR Compac. Results obtained indicated that the assay detects antibodies specific of PPRV at days 7 p.v. and p.i. in 85% of cases.

### COMPOSITION OF THE KIT

- Microtitration plates of 96 wells
- Vials with Positive Control
- Vials with Negative Control
- Vials with Peroxidase Conjugate
- Bottle with Washing Solution
- Bottle with diluent
- Bottle with stop solution
- Bottle with substrate (ABTS) ready to use

PRODUCT MANUFACTURED BY INGENASA  
REGISTRATION NUMBER: 3780 RD

SHELF LIFE: **12 months**  
Store at 2°C-8°C