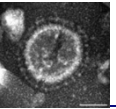
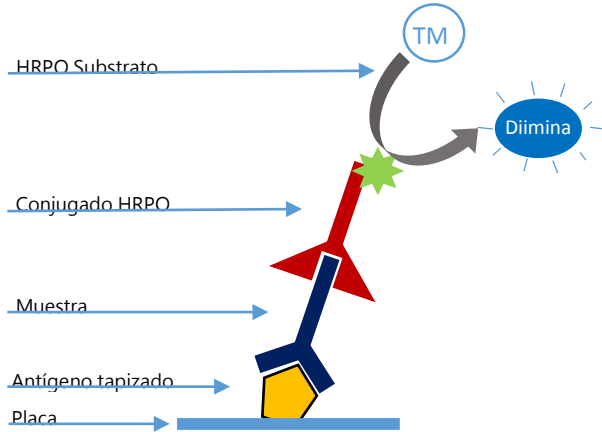
## INgezim PEDV

R.11.PED.K1

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

## INgezim PEDV

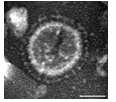R.11.PED.K1

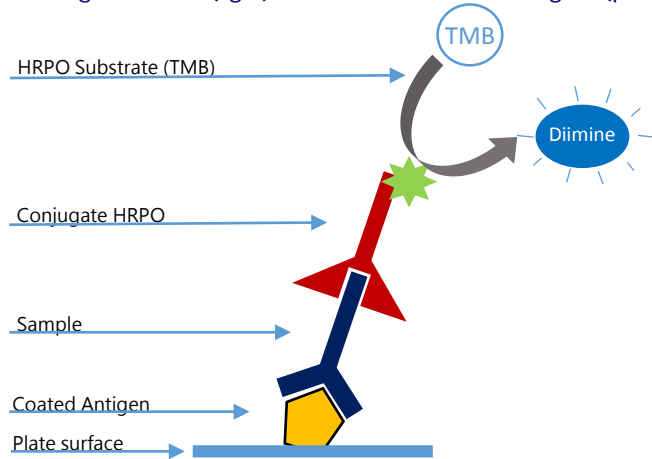**INgezim PEDV** is based on the indirect ELISA technique, that uses a monoclonal antibody (MAb) specific to porcine immunoglobulines (IgG), and recombinant antigen (protein S of Porcine Epidemic Diarrhea Virus, PEDV).

### TECHNICAL BASIS OF THE KIT

1. Plates are coated with PEDV antigen (protein S of PEDV). Serum samples are added and incubated.
2. If the samples contain antibodies specific to PEDV, they will bind to the antigen.
3. When a MAb-PO specific of porcine immunoglobulines is added, it will bind to the IgG bound to the antigen. The binding is detected by the development of a colorimetric reaction after the addition of the substrate.

### APPLICATION

Detection and /or titration of specific antibodies to PEDV in porcine serum samples.

### INTERPRETATION OF THE RESULTS

Samples with a S/P ratio equal or higher than 0.35 are considered **Positive** and samples with a S/P ratio lower than 0.35 are considered **Negative**.

### DIAGNOSTIC SENSITIVITY AND SPECIFICITY

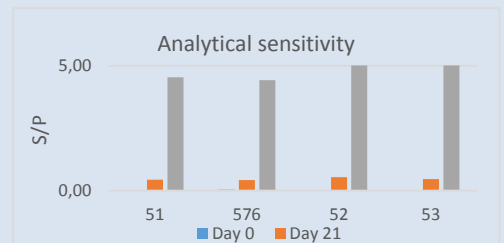
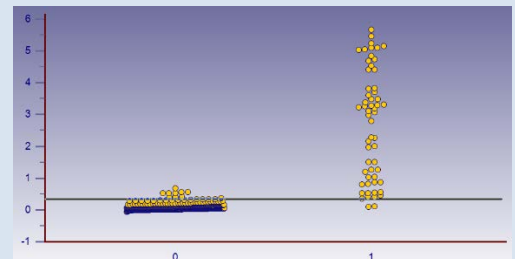
In order to evaluate the performance of the assay concerning to diagnostic sensitivity and specificity, the following sera were analysed:

- 894 negative sera
- 57 European and American sera, positive by different techniques.

Results indicated that the assay shows **94.74 sensitivity** and **98.89 % specificity**.

### ANALYTICAL SENSITIVITY

Four animals were experimentally vaccinated and bled at days 0, 21 and 37 p.v. Results obtained indicated that the assay was able to detect at day 21 p.v. the 4 animals as positive.

### EXTERNAL STUDIES

The study consisted on compare two commercial kits: Ingezim PEDV (INGENASA) y PEDV Test kit (BioVet). 19 samples of Russian pigs from negative and positive (with and without clinical signs) farms, were analyzed. The results are shown below:

FARM STATUS	CLINICAL SIGNS	PEDV Test Kit (p/n)*	INgezim PEDV (p/n)
Free	No	1/5	0/6
	Yes	6/0	6/0

\*p/n: n° positive / n° negative

### COMPOSITION OF THE KIT

- Microtitration plates of 96 wells coated with Ag
- Vials with Positive Control
- Vials with Negative Control
- Vials with Peroxidase Conjugate
- Bottle with Washing Solution
- Bottle with Diluent
- Bottle with Substrate (TMB)
- Bottle with Stop Solution

PRODUCT MANUFACTURED BY INGENASA  
REGISTRATION NUMBER: 3814 RD

SHELF LIFE: **12 months**  
Stored at 2°C-8°C

Ed.020217